TRACHEAL-KOMPRESSEN

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PZN -4770462

TRACHEAL COMPRESS

Tracheal Compresses are meant for cannula wearers. The relatively light fleece pad is aluminium vapoured on one side. Secretion is well soaked up. Soaking effect is only sufficient in case of normal or slight secretion. Otherwise the use of the more absorbent **2-KAM®** Tracheal Compress or of the **3-KAM®** Tracheal Compress is recommendable. The **Tracheal Compress** is unsterile and meant for single use only. It should be replaced several times every day.

The tracheal cannula is pushed through the existing circular opening of the **Tracheal Compress** namely so, that the aluminium vapoured side rests against the skin. The aluminium vapoured coating prevents a sticking of the compress to the skin.

ATTENTION!

Should appear any skin irritations (e.g. reddening, itching) the product has to be removed immediately. Possibly a special skin sensitivity or allergic conditioned incompatibility in this case is existing. Please contact your doctor and use an aluminium-free compress in the meantime.

Don't use this product during a radiation treatment: Danger from skin damages!

ANDREAS FAHL

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August-Horch-Straße 4 a · D-51149 Köln – Germany

Phone +49 (0) 22 03/29 80-0 · Fax +49 (0) 22 03/29 80-100

Contents: 10 pcs.

Size: 8 x 10 cm

REF 30000